

Instructions For Use KT024-IFU

Rev. Date: June 12, 2018

Revision: 6

Page 1 of 2

626 Wilshire Blvd, Suite 410 Los Angeles, CA 90017 - Tel. +1 (408) 580-1396 www.genebiosolution.com

Mucicarmine Stain Kit

Description: Mucicarmine Stain Kit is intended for use in the histological visualization of acid mucopolysaccharides in tissue sections. This product is useful in distinguishing mucin negative undifferentiated squamous cell lesions from mucin positive adenocarcinomas. In addition, this product will stain the mucopolysaccharide capsule of *Cryptococcus neoformans*.

Mucin:

Capsule of *Cryptococcus*:

Pink/Red

Nuclei:

Red

Other tissue components:

Blue to Green

Yellow

Uses/Limitations: For In-Vitro Diagnostic use only.
Histological applications.
Do not use past expiration date.
Use caution when handling these reagents.

Control Tissue: Colon
Intestine
Bronchi Epithelial cells

Availability/Contents:

<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
Hematoxylin, Mayer's (Lillie's)	125 ml	15-30°C
Bluing Reagent	125 ml	15-30°C
Mucicarmine Solution	125 ml	2-8°C
Tartrazine Solution	125 ml	15-30°C

Precautions: This product is a single-use, non-sterile, in vitro diagnostic device.
Keep away from open flame.
Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.
Use in chemical fume hood whenever possible.



Gene Biosolution

626 Wilshire Blvd, Suite
410 Los Angeles, CA
90017

Tel: +1 (408) 580-1396

Instructions For Use KT024-IFU

Rev. Date: June 12, 2018

Revision: 6

Page 2 of 2

626 Wilshire Blvd, Suite 410 Los Angeles, CA 90017 - Tel. +1 (408) 580-1396 www.genebiosolution.com

Procedure (Standard):

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. If sections are Zenker-fixed, remove mercuric chloride crystals using iodine and clear with sodium thiosulfate. Rinse in running tap water.
3. Stain slide in Hematoxylin for 2-3 minutes.
4. Rinse for 2 minutes in running tap water.
5. Apply Bluing Reagent for 10-15 seconds.
6. Rinse slide in distilled water.
7. Stain slide in Mucicarmine Solution for 10 minutes.
8. Rinse quickly in tap water followed by a quick rinse in distilled water.
9. Stain slide in Tartrazine Solution for 1 minute.
10. Rinse quickly in tap water followed by a quick rinse in distilled water.
11. Dehydrate through graded alcohols.
12. Clear, and mount in synthetic resin.



Gene Biosolution

626 Wilshire Blvd, Suite
410 Los Angeles, CA
90017

Tel: +1 (408) 580-1396