

Instructions For Use KT037-IFU

Rev. Date: Dec. 13, 2018	Revision: 4	Page 1 of 2
--------------------------	-------------	-------------

626 Wilshire Blvd, Suite 410 Los Angeles, CA 90017 - Tel. +1 (408) 580-1396 www.genebiosolution.com

Picro-Sirius Red Stain Kit (For Collagen)

Description: The Picro-Sirius Red Stain Kit (For Collagen) is intended for use in the histological visualization of muscle and collagen fibers in tissue sections. The PSR stain may be viewed using standard light microscopy or polarized light resulting in birefringence of collagen fibers.


Light Microscopy

<u>Collagen:</u>	Red
Muscle Fibers:	Yellow
Cytoplasm:	Yellow

Polarized Light Microscopy

Collagen: Yellow-Orange and Green Birefringence

Uses/Limitations: Not to be taken internally.
 For In-Vitro Diagnostic use only.
 Histological applications.
 Do not use if reagents become cloudy.
 Do not use past expiration date.
 Use caution when handling reagents.
 Protocol has not been tested on and may not be suitable for frozen sections.
 Non-Sterile.

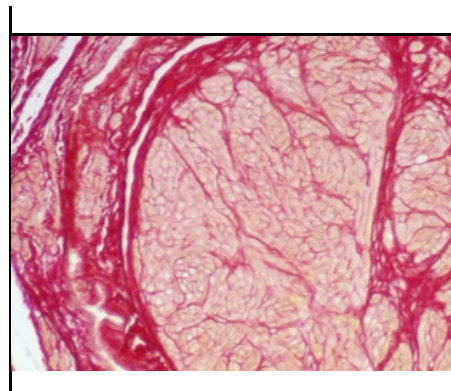

Control Tissue: Lung
 Uterus
 Muscle
 Kidney

Kit Contents:

<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
Picro-Sirius Red Solution	250 ml	15-30°C
Acetic Acid Solution (0.5%)	250 ml x 2	15-30°C

Precautions: This product is a single-use, non-sterile, in vitro diagnostic device. Avoid contact with skin and eyes. Harmful if swallowed. Follow all Federal, State, and local regulations regarding disposal. Use in chemical fume hood whenever possible.



Gene Biosolution

626 Wilshire Blvd, Suite
 410 Los Angeles, CA
 90017

Tel: +1 (408) 580-1396

Instructions For Use
KT037-IFU

Rev. Date: Dec. 13, 2018

Revision: 4

Page 2 of 2

626 Wilshire Blvd, Suite 410 Los Angeles, CA 90017 - Tel. +1 (408) 580-1396 www.genebiosolution.com

Procedure:

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Apply adequate Picro-Sirius Red Solution to completely cover tissue section and incubate for 60 minutes.
3. Rinse slide quickly in two changes of Acetic Acid Solution (0.5%).
4. Rinse slide using absolute alcohol.
5. Dehydrate in 2 changes of absolute alcohol, clear, and mount in synthetic resin.



Gene Biosolution

626 Wilshire Blvd, Suite
410 Los Angeles, CA
90017

Tel: +1 (408) 580-1396